

MXN95R1

Respirador de Partículas N95 Particulate Respirator N95



SAFETY PRODUCTS
MANUFACTURING

Instrucciones de colocación



1. Sostenga el respirador frente a su rostro soportándolo en la palma de su mano; deje que la cinta elástica cuelgue libremente. Lleve la parte inferior de la cinta sobre su cabeza y ajústela alrededor del cuello de manera cómoda y holgada.



2. Coloque el respirador sobre el rostro tomando como referencia inicial la barbilla, para un mejor ajuste. Lleve la cinta superior por encima de su cabeza y colóquela en la parte posterior por encima de sus orejas.



3. Ajuste la tensión de la cinta, tanto en la parte superior como inferior, de manera que se coloque cómodamente sin permitir fugas. El respirador viene pre formado en la copa nasal para garantizar el sello de forma natural.



4. Coloque las manos cubriendo el respirador e inhale profundamente. Al respirar deberá sentir una presión negativa, si esto sucede o si hay fugas, vuelva a colocarlo y ajuste las bandas para obtener un mejor sello.

Placement Instructions



1. Hold the respirator in front of your face in the palm of your hand and let the elastic strap hang freely. Bring the lower part of the strap over your head and fit it around your neck comfortably and loosely.



2. Position the respirator on your face using the chin as an initial reference for a better fit. Wear the top strap over your head and place it at the back above your ears.



3. Adjust the tension of the strap, on both the top and bottom, so that it fits comfortably without allowing leaks. The respirator with a pre-shaped shell to ensure a natural seal.



4. Place your hands covering the respirator and inhale deeply. A negative pressure should be felt when breathing; if this isn't the case or if there is an air leak, reposition the respirator and adjust the straps for a better seal.

WWW.SAFETYPRODUCTSMFG.COM

CERTIFICADO COFEPRIS NO. 0564C2020 SSA NOM-STPS-116-2009

ELABORADO POR: ALUMITEK INGENIERÍA S DE RL DE CV

RFC: AIN161221FX0 CALLE DE LA PRESA 1606 INT. 5 COL. FUENTES DEL VALLE C.P.32500
CD. JUÁREZ CHIHUAHUA, MÉXICO